

COMPRESSOR NEBULIZER



Instruction for Use

 In case of any serious incident with the device, immediately contact the manufacturer and national competent authorities.

Use the device only as described in this Instruction for Use and consult your physician in case of doubts.

4. PRECAUTIONS

Rules of use the Nebulizer

Check in the medicine package leaflet for possible contraindications for use with common aerosol therapy systems.

This diagram shows the exploded view of the vacuum cleaner's main body and motor assembly. The components are labeled as follows:

- 1**: Motor housing / main body.
- 2**: Motor.
- 3**: Motor mounting bracket.
- 4**: Motor mounting screws.
- 5**: Motor mounting plate.
- 6**: Motor mounting bracket (bottom).

Insert the mask or mouthpiece directly onto the nebulizer.

Use of mouthpiece is recommended in treatment of lower respiratory tract in grownups and children above 5 years old.

Use of masks:
Put on a mask in the way that it will close your nose and mouth, and conduct inhalation of the drug. Inhale and exhale through the

When the treatment has been completed, switch the unit off by setting the switch into the «0» position and disconnect the plug from the wall socket.

⚠ WARNING! The device can work for 30 min without a pause, than you should let the device to cool down for 30 min.

⚠ WARNING! The use of dirty filter or filter made of another material, for example, cotton, may lead to device damage.
Use original filters only.

- ⚠ WARNING!** Air filters cannot be cleaned or washed.

Follow carefully the cleaning and disinfection instructions of the spare parts as they are very important for the device performances and the therapy success.

- Opening of the device is prohibited. Repair of the device should be conducted only in technical service centers recommended by B. Well company. Only use B. Well original spare parts. Original spare parts can be purchased from your local distributor.

Maintenance and storage


- do not store the device at extremely high or low temperatures, high humidity or under direct sunlight;
- do not bend or wrap the air tube.

Contact an authorized Customer Service Centre in case you need help about the use and the maintenance of the equipment.
In case of any malfunction look at the table and try to eliminate it.

Problems	Solutions
The device does not switch on	- Make sure the power plug is firmly fitted to the wall socket. - Make sure that the device has been operating within operating limits indicated in this Instruction for Use (30 min ON / 30 min OFF).
The device does not nebulize or nebulizes weakly	- Make sure that the ends of the air tube are tightly fitted to the main unit and nebulizer. - Verify that the nebulizer is not empty or has been filled with the proper quantity of medicine (max 8 ml). Verify that the nebulizer nozzle is not obstructed.

In case of failure, address to qualified personnel authorized by B.Well. Do not open the device in any case. The unit has no user-serviceable parts within and does not need internal maintenance or lubrication.

Nebulization rate: 0.4 ml/min.

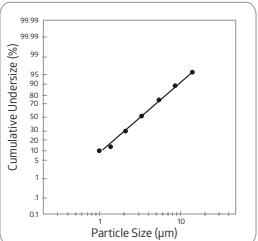

Storage conditions:

Humidity:	MIN 10% RH – MAX 95% RH
Atmospheric pressure:	700 hPa – 1060 hPa
Temperature:	MIN -25 °C – MAX +70 °C
Humidity:	MIN 10% RH – MAX 95% RH
Atmospheric pressure:	700 hPa – 1060 hPa

Nebulizer kit, air tube, mouthpiece, nosepiece, masks – 6 months (for 2 times per day usage) to 1 year (one time per day usage).
Air filter – 200 working hours or 1 year.

- The information defined above is advisory and can change depending on physical properties (temperature, viscosity, density) of the nebulized substance.

This device fulfils the provision of the EC directive (EU) 2017/745 Medical Device Regulation and the European Standard EN 13544-1:2007+A1:2009 Respiratory therapy equipment – Part 1: Nebulizing systems and their components.

The warranty does not apply to damage caused by using adapters not recommended by the B.Well company and the overvoltage. When a manufacturing defect is revealed during the warranty period a faulty unit would be repaired or, if repairing is impossible, replaced with another one.

	READ INSTRUCTION BEFORE USE	IP21	HOUSING INGRESS PROTECTION RATE
	DISPOSAL FOR SEPARATE COLLECTION	REF	ARTICLE NUMBER
	TYPE BF EQUIPMENT		MANUFACTURER'S NAME
	CLASS II EQUIPMENT	10°C - 40°C OPERATION CONDITION TEMPERATURE -25°C - 5°C STORAGE CONDITION TEMPERATURE	CE MARK 0044 UNIQUE DEVICE IDENTIFICATION UDI
SN	SERIAL NUMBER	EC REP	EC REPRESENTATIVE
LOT	LOT NUMBER		
MD	MEDICAL DEVICE		

B.Well Swiss AG
Bahnhofstrasse 24, 9443 Widnau, Switzerland
www.bwell-swiss.ch