

WC-150


NASAL ASPIRATOR



EN Instruction for Use

1. INTRODUCTION

Thank you for purchasing the B.Well nasal aspirator WC-150. B.Well nasal aspirator is able to operate with strong power for keeping nose clean and clear, comparing with the traditional rubber aspirator. This device is designed for more safety and easier operation and compact for home and travel use.

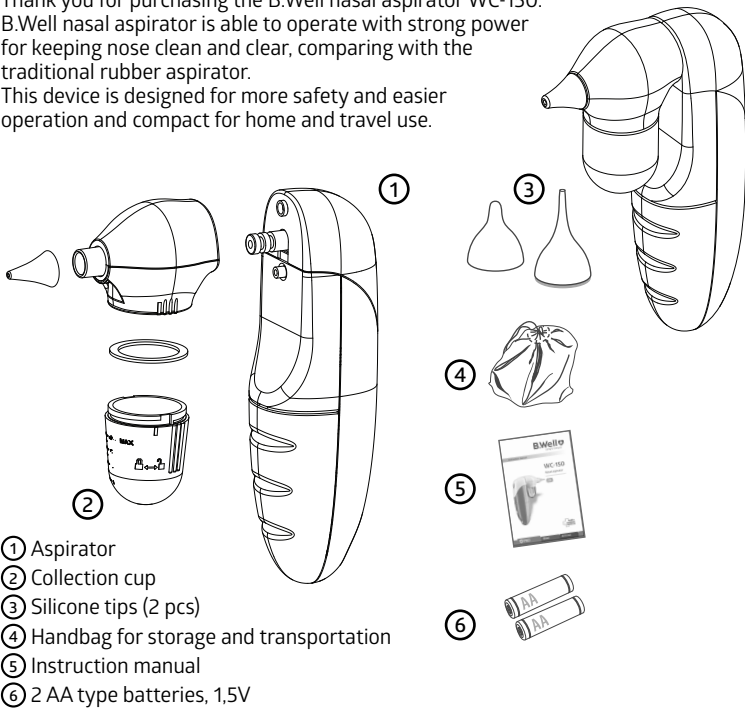

2. PRODUCT FEATURE

1. Strong power:

Put aspiration tip into nostril, then you may use this device to remove nasal secretions continuously or intermittently.

2. Safe:

The soft aspiration tip is for comfortable design to prevent the possibility from injuring when using this device.

3. Compact and easy design:

The device is compact, easy to hold and operate. It is also safe and comfortable for baby use.

4. Battery:

This device only needs two 1.5V AA batteries, it is convenient to change the batteries anytime, anywhere. Low battery indicator.

5. Music:

Instead of the dull/noising sound when using the device, our device provides the unique design of the chord ring and 12 worldwide children's ballads. With the beautiful songs, the baby will feel more comfortable treated with this device.

6. Washable suction head:

Tips, collection cup and head part can be taken off from the device and washed. Collection cup can also be disinfected by boiling water.

3. INTENDED PURPOSE

The nasal aspirator is designed for using intermitten suction to remove nasal secretion and mucus in children (age 0–12 years old) at home environment.

Exact medical indication:

The removal of nasal secretions and mucus includes addressing nasal congestion, mucus buildup, and blocked nasal passages in infants and children.

Intended user:

The nasal aspirator is designed for use by laypersons, typically parents or caregivers, in a home environment. We define two main user conditions, as follows:

- 1) Knowledge:
 - Minimum:
 - Can distinguish: mouth, ear canal, nasal cavity.
 - Understands hygiene.
- 2) Language understanding:
 - Be able to read the local official language. (For understanding the IFU.)

In case of any serious incident with the device, immediately contact the manufacturer and national competent authorities.

Contraindications:

Users with damage to the integrity of the mucous membrane of the nasal passages and some individual features of the structure of the nasal septum.

4. PRECAUTIONS

1. Do not push tip deep into baby's nostril. Use the nasal aspirator in the recommended direction. Please keep the nostril toward down.
2. For the baby's security, aspiration time is limited to few seconds.
3. Do not use the nasal aspirator in case of nasal irritation or hemorrhage.
4. In the case of infection, fever, nasal irritation or hemorrhage, please consult a doctor.
5. Do not use the nasal aspirator without a silicone tip or with a deteriorated collection cup.
6. Wash silicone tip and collection cup after each use.
7. Wash your hands before and after using the nasal aspirator.
8. Do not modify this equipment without authorization of the manufacturer.
9. Do not expose the device to extreme temperature, very high humidity level or direct sunlight.
10. Avoid extreme stress, shock or accidental drop-down.
11. Do not wash the body of the device with water or any kind of liquids. Strongly recommended wipe the device with soft cloth and keep the device clean.
12. Please replace the batteries when the power or/and the sound of the pump getting weak during using the device.
13. It is recommended to remove the batteries if the device will not be used for a period of time.
14. It is recommended to use this device intermittently. Press the power button to generate vacuum for removing nasal secretions. To stop the suction, release the power button.
15. The nasal aspirator is intended for cleaning/removing the nasal secretions only. Do not use the device other than the designated purposes.
16. When using the device, parents should alert children from holding their breath or swallowing, and should keep their mouth open to avoid pressure imbalance in the ear.
17. If the user does not feel well when using the device, stop and consult the doctor immediately.
18. Keep out of reach of children when not in use to prevent children from accidentally swallowing components.
19. Store the unit in an area away from sunlight, dust and humidity.
20. It is recommended not to use rechargeable, unqualified or different spec of batteries as it may damage the device or cause circuit shortcut.
21. Under any circumstances, the device is ONLY for removing nasal secretion. Any further medical action should follow the device of doctors.
22. Do not apply excessive force when placing the aspiration tip into the nostril.
23. This device is for home use only. Please consult a doctor in case of nasal diseases. Clean the aspirator tip prior each use.
24. The nasal aspirator is not a toy. The intended operators for this device shall be parents or adults who can follow the instructions and notices in Instruction manual to operate the device. The device is not allowed to be used by children themselves.
25. Do not use the nasal aspirator when patients have the following contraindications:
 - Nasal bleeding or any open wounds inside
 - Acute head, facial or neck injury
 - Coagulopathy or bleeding disorder
26. Before using the device, please read the Instruction manual carefully. Wrong use of the device may cause the following hazards/complications:
 - Mechanical trauma
 - Nasal irritation/bleeding
 - Mucosal hemorrhage
 - Discomfort and pain

NOTE: This nasal aspirator must only be used with B.Well WC series tips. Please contact the retailer to buy those accessories.

5. BATTERIES INSTALLATION

1. Remove the battery cover as shown.
2. Insert two 1.5V alkaline AA size batteries matching positive (+) to positive (+) and negative (-) to negative (-).
3. Press down on batteries to make sure they are completely inserted.
4. Replace the battery cover as shown.

NOTE:



- When changing the batteries, use batteries of the same type, make and capacity.



- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Use batteries free from heavy metals.
- Disposal information:** Should you wish to dispose of the article, do so in accordance with current regulations. Details are available from your local authority. The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

6. PRODUCT INSTRUCTION

1. Choose the correct tip to use, according to shape and size of the nostrils of your baby.
 - The large tip is recommended for use when the mucus is in a more solid form.
 - The narrow tip is recommended for use when the mucus is loose and in a liquid form.
2. Be sure to place the tip securely on the aspirator. Make sure tip is completely over the rib before using.

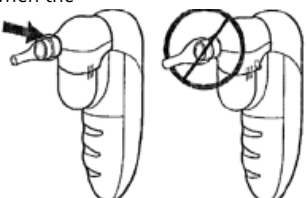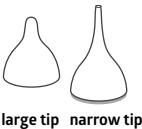





3. Put aspiration tip into nostril.

4. Press and hold the power button to allow the suction to clear the mucus.

NOTE: The mucus may not immediately appear in the collection cup.

After each use it is recommended to do a quick clean of the tip by removing the tip from the unit.

5. Please keep the nostril toward down when using the device to make the device operate efficiently. Be sure to keep the tip of the aspirator at a 90° angle. Do not point it up into the nostril. Allow the suction to clear the mucus. Repeat in other nostril and as needed. Release power button to stop suction.

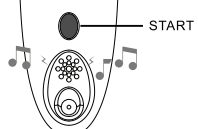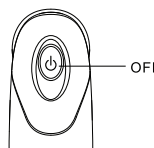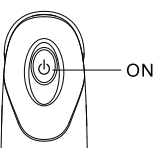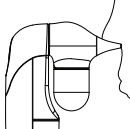

7. CLEANING INSTRUCTIONS

After using this device, the nasal secretion will be kept in the collection cup. It is recommended that the silicone tip and collection cup to be rinsed with hot water between uses to prevent mucus from drying and clogging the unit.

- Separate the tip and the head module from the base of the aspirator by gently pulling it apart.
- Unscrew collection cup by turning it clockwise from the head module.
- Body device cleaning: wipe the unit with a clean soft dry cloth.

NOTE: Remove the connection module immediately after using the device. Do not put the aspirator horizontally unless the collection cup is disconnected. Otherwise liquid from the connection module can get inside the device and cause damage.

NOTE: DO NOT wash the body of the device with water or any kind of liquids. Do not clean the unit with solvents or abrasive products.

- Cleaning of collection cup, head module and aspiration tip: washing the collection cup, head module and aspiration tip with mild, non-abrasive soap in water for 3 minutes. Then rinse these parts with running hot tap water for 3 minutes and air dry. After the cleaning process these parts shall be disinfected by boiling water.

8. SPECIFICATIONS

Effective vacuum pressure	400 mmHg (53.3 kpa)
Battery	Two 1.5V AA alkaline batteries
Sound pressure level	≤75dB
Weight	250 g
Dimensions	93,5×39,9×148 mm
Aspiration tip	Soft silicone
Motor	3V/DC
Operation condition	Temperature: +16°C to +35°C Humidity: 15% to 85% (non-condensing) Atmospheric pressure: 70kPa to 106kPa
Storage condition	Temperature: -25°C to +55°C Humidity: 15% to 85% (non-condensing) Atmospheric pressure: 70kPa to 106kPa

9. WARRANTY

Warranty period is 2 years from the date of purchase. This warranty doesn't cover any damages caused by improper using, and also battery, storage bag, packaging, the accessories supplied with the device and those parts subject to normal wear and tear. When a manufacturing defect is revealed during the warranty period a faulty unit would be repaired or, if repairing is impossible, replaced with another one. The manufacturer may change units partially or completely if necessary, without prior notice. *Manufacturing date is encoded in the SN number at the device: first two numbers are numbers of the week, second two numbers are last numbers of the year.*

10. SYMBOL INFORMATION

	FOLLOW INSTRUCTIONS BEFORE USE		KEEP DRY
	HOUSING INGREE PROTECTION RATE: IP22 (Protected from the penetration of solid bodies with dimension greater than 12,5 mm. Protected from the penetration of vertically falling water drops)		KEEP AWAY FROM SUNLIGHT
	OPERATING CONDITION: 16°C ~ 35°C		STORAGE CONDITION, TEMPERATURE: -25°C ~ 55°C
	DISPOSAL FOR SEPARATE COLLECTION		STORAGE & OPERATION CONDITION, HUMIDITY 15% to 85%
	CE MARK		OPERATING CONDITION, ATMOSPHERIC PRESSURE 70kPa to 106kPa
	SERIAL NUMBER		BF TYPE DEVICE
	REF NUMBER		UNIQUE DEVICE IDENTIFICATION
	MANUFACTURER'S NAME		MEDICAL DEVICE
	INFORMATION		EC REPRESENTATIVE
	WARNING		

Last revision 2024-W27

B.Well Swiss AG
Bahnhofstrasse 24, 9443 Widnau, Switzerland
www.bwell-swiss.ch